




---

### YOUR MEDICATION

Name: \_\_\_\_\_  
 Concentration: \_\_\_\_\_%  
 Total Volume: \_\_\_\_\_ml  
 Your bolus button lock-out period is \_\_\_\_\_ minutes.

---

### CONTACT

Doctor: \_\_\_\_\_  
 Phone: \_\_\_\_\_  
 Next Appointment:  
 Date: \_\_\_\_\_ Time: \_\_\_\_\_

---

### SYMPTOMS TO WATCH FOR

Immediately report any of these symptoms to your healthcare provider:

- Increase in pain
- Redness, swelling, pain or discharge at incision or catheter site
- Metallic taste in your mouth
- Dizziness
- Drowsiness
- Other side effects: \_\_\_\_\_

Special Instructions: \_\_\_\_\_

---

### PRECAUTIONS

- Do not reuse the ACCUFUSER pump
- When taking a shower, place the ACCUFUSER pump outside of the shower or in a waterproof bag
- Store the ACCUFUSER pump at room temperature
- Protect the ACCUFUSER pump from sunlight

Visit our website: [www.controlpostoppain.com](http://www.controlpostoppain.com)  
 Clinical Support: (800) 806-8164



Manufactured for and distributed exclusively by:  
 Curlin Medical, Inc., 15751 Graham Street  
 Huntington Beach, CA 92649 USA 1 (888) 287-5999

©2006 by Curlin Medical, Inc. All rights reserved.  
 ACCUFUSER and the Curlin Medical logo are registered trademarks, and The Beauty of Personal Post-Op Pain Control and Curlin Saturation Catheter are trademarks of Curlin Medical, Inc. U.S. Patents: 6,024,724; 6,689,110B2; 6,676,643B2.  
 PN 430-9044 Rev. A 11/06



# ACCUFUSER®

## POST-OP PAIN CONTROL PUMP



### ABOUT YOUR ACCUFUSER POST-OP PAIN CONTROL PUMP

Information for the Patient


### THE BEAUTY OF PERSONAL POST-OP PAIN CONTROL™

Your doctor has prescribed the ACCUFUSER® Post-Op Pain Control Pump to help relieve your pain and provide you a comfortable recovery from surgery. The ACCUFUSER is a small balloon-like pump filled with non-narcotic pain medication. A tiny tube called a catheter was inserted near the incision during surgery. The ACCUFUSER pump will deliver continuous pain medication through the catheter directly to the site of your incision. Your doctor may have prescribed for you an ACCUFUSER pump that comes with a push-button control called a bolus button. You can use the bolus button to deliver a dose of additional pain medication from time to time if needed.




# ACCUFUSER®

## POST-OP PAIN CONTROL PUMP

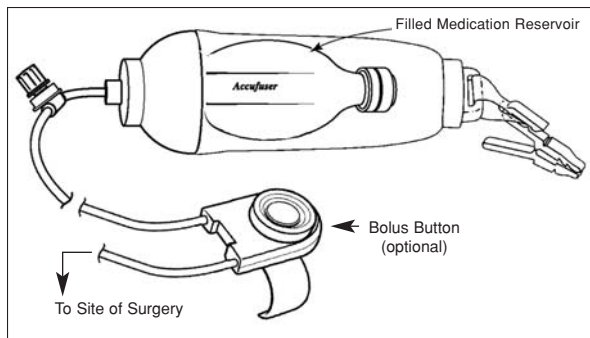


### TO START MEDICATION DELIVERY

Start and stop the ACCUFUSER pump only as directed by your doctor or healthcare provider. To start continuous medication delivery, open the on/off clamp on the ACCUFUSER tubing. (The clamp is open when the tubing can move freely through the eye of the clamp.)

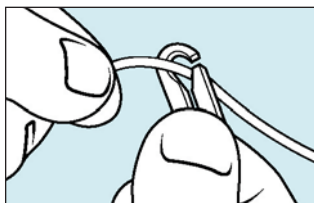


### TO STOP MEDICATION DELIVERY

To stop medication delivery, close the on/off clamp on the ACCUFUSER tubing. (The clamp is closed when the tubing is tightly pinched by the lower half of the clamp.)

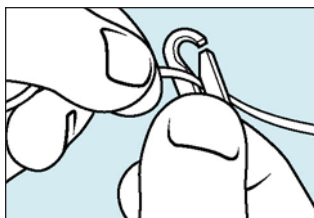

### DURING MEDICATION DELIVERY

Wear your portable ACCUFUSER pump by attaching it to your clothing with the clip or placing it in the carry pouch.

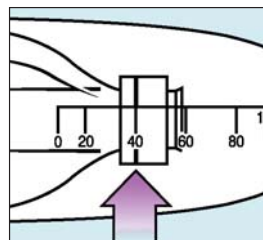

As the pump delivers your medication, the balloon reservoir inside will slowly become smaller. You can track the progress of medication delivery by noting the position of the reservoir's flow indicator on the number scale on the pump.


During medication delivery, check for the following:

- The ACCUFUSER pump is emptying
- There is no leakage from the ACCUFUSER system
- Air has not entered the medication reservoir
- The tubing is not kinked
- All system connections are secure

If you notice any problems, contact your doctor or healthcare provider immediately.

### USING THE BOLUS BUTTON

If your ACCUFUSER pump is equipped with a bolus button, you can use it to receive a dose of additional pain medication when you need it.

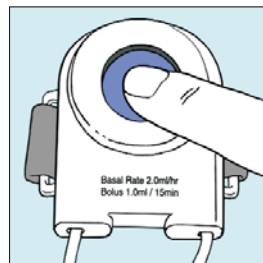


Secure the bolus button directly on your skin. A strap is provided so that you can wear it on your wrist.

To receive a dose, press and hold the bolus button for a few seconds. The button will feel "stiff" to the pressure of your finger. This is normal and indicates that the pump is delivering the medication. You'll know when the dose has been delivered when the button feels "spongy" and easy to push.

Each time you press the button to receive a dose, a lock-out period begins. You cannot receive another dose until the lock-out period is over. During the lock-out period the bolus button feels "spongy" and easy to push. Refer to the back of this brochure where your healthcare provider has written down how long your lock-out period lasts.

### WHEN MEDICATION DELIVERY IS COMPLETE


Medication delivery is complete when the flow indicator on the pump is at the "0" mark on the scale. This means there is no medication left in the reservoir. Unless you have been given other instructions, your doctor or healthcare provider will remove your ACCUFUSER catheter.

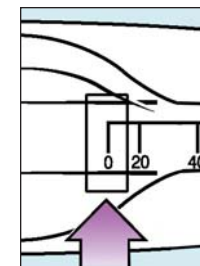


### TO REMOVE THE CATHETER

If your doctor or healthcare provider has instructed you to remove the ACCUFUSER catheter upon completion of medication delivery, follow their instructions while keeping in mind the following key steps. If you have more than one catheter insertion site, repeat the steps for each catheter.


1. Remove the dressing covering the catheter site.

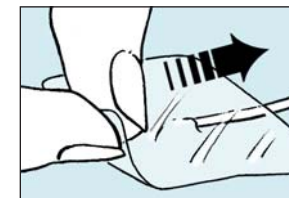

2. Grasp the catheter close to the skin and gently pull it out. It should be easy to remove and not be painful. If it is difficult to remove, or it stretches, or removal is painful, STOP and call your doctor or healthcare provider. DO NOT cut or pull hard to remove the catheter.

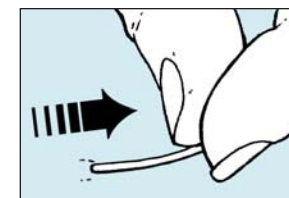

3. After you remove the catheter, place a bandage over the catheter site.

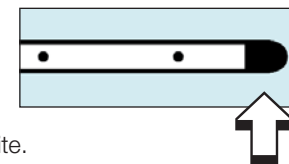

Check to see if there is a black mark at the end of the catheter. If you do not see a black mark, call your doctor immediately.

4. If there is redness or warmth at the catheter site, call your doctor.